## REGULATEUR DE VIDE / VACUUM REGULATOR

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- **Capacity**: 100 ml as standard  
150 ml on demand

- **Mechanical overflow safety system**

- **Anti-bacterial filter**:

**Current model**: Anti-bacterial plastic filter up-front for RVTM2 vacuum regulator.

**Previous model**: Anti-bacterial paper filter for RVTM vacuum regulator.

See on page 9 the technical data of the anti-bacterial filter.

- **Cleaning**:

1/ RVTM2 model with anti-bacterial plastic filter up-front:

Change the filter after each patient or, in case of a long stay patient, when necessary after inspection of the filter clogging level. Pull the filter with a rotating movement and throw it away respecting the safety rules. Put a new filter by pushing it until ratchet mechanism.

**The safety jar being protected against infections by the filter up-front, it is not necessary to disinfect it any longer except in case of occasional liquids' overflow or perforated filter** (see below for disinfection procedure).

2/ RVTM model with anti-bacterial paper filter:

Change the filter after each patient or, in case of long stay patient, when necessary, after inspection of the filter clogging level. Each time you change the filter it is essential to disinfect the safety jar according to the procedure below:

Disinfection procedure:

- Unscrew the safety jar and disassemble the cover from the jar.
- Dump the bottle in a decontaminating and cleaning solution.
- Clean, rinse thoroughly, let soak for 1 minute in clean water and dry.
- Sterilize or disinfect.
- Put a new filter at the right place.
- Re-assemble the parts together and re-screw the safety jar on the vacuum regulator.

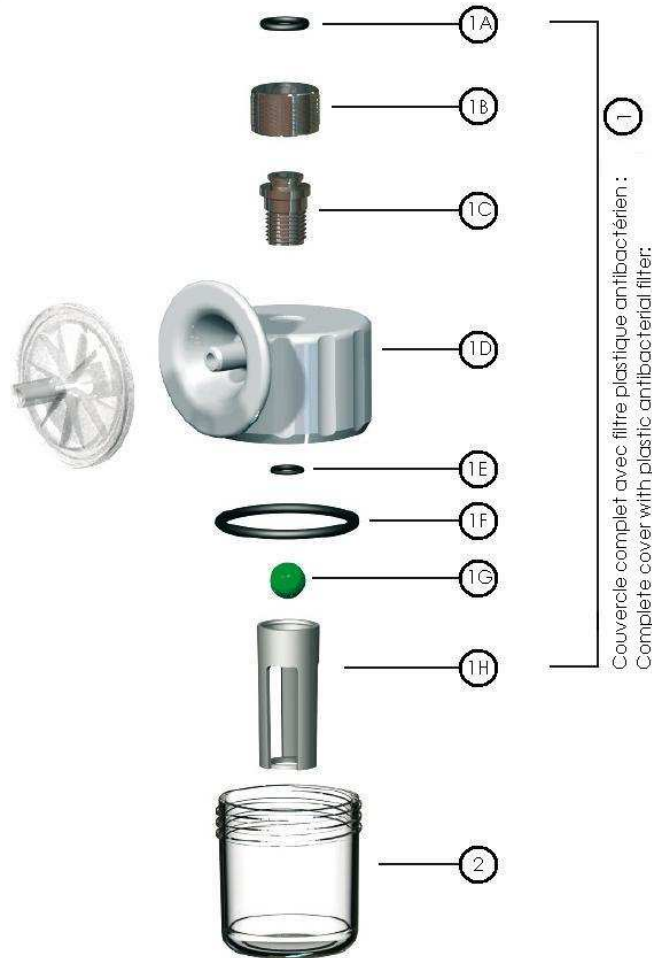
**The safety jar withstands the autoclave up to 134°C.**

Caution: the compatibility of the cleaning products used with polycarbonate must be checked. For information, products containing amphoteric components, glutaraldehyde, formaldehyde, glyoxal, succinic acids, **are not compatible** with polycarbonates. Also avoid solutions for floors and surfaces.

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• **Technical view**: Safety jar with plastic anti-bacterial filter up-front

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
No	Ref.	Description	Material
<b>1</b>	<b>11772</b>	<b>COMPLETE COVER WITH PLASTIC ANTIBACTERIAL FILTER UP-FRONT</b>	
1A	11781	1/2G inlet gasket	Ethylene Propylene Diene Monomer
1B	11845	1/2G nut	Brass
1C	11844	Inlet stem	Brass
1D	11770	Cover only with filter holder	Polypropylene
1E	11701	O-ring for safety cage	Polypropylene Ethylene Propylene Diene Monomer
1F	11780	O-ring for safety jar	Polypropylene Ethylene Propylene Diene Monomer
1G	11698	Safety overflow ball	Polypropylene
1H	11697	Safety overflow cage	Polypropylene
<b>JARS ONLY</b>			
2	17630	100 ml jar only	Polycarbonate
	11731	150 ml jar only	Polycarbonate

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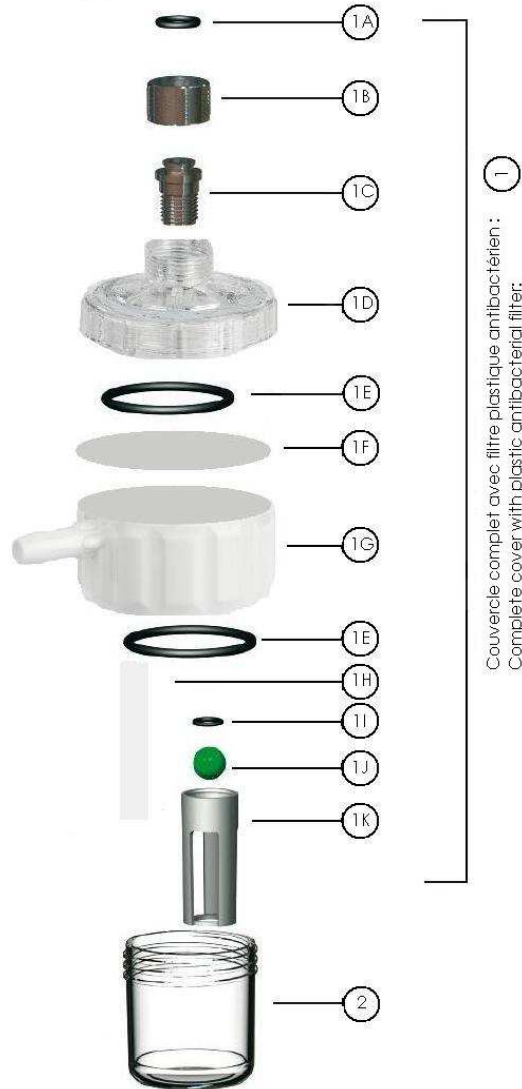• **Technical view**: Safety jar with paper antibacterial filter in the cover

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No	Ref.	Description	Material
1	11741	<b>COMPLETE COVER WITH PAPER ANTIBACTERIAL FILTER IN THE COVER</b>	
1A	11781	1/2G inlet gasket	Ethylene Propylene Diene Monomer
1B	11845	1/2G nut	Brass
1C	11844	Inlet stem	Brass
1D	11743	Filter translucent cover only	Polycarbonate
1E	11780	O-ring for safety jar	Polypropylene Ethylene Propylene Diene Monomer
1F	11817	Paper antibacterial filter	Borosilicate fibreglass + laminated polyester
1G	11744	Safety cover only	Polycarbonate
1H	11811	Diving Tube	Polycarbonate
1I	11701	O'ring for safety cage	Polypropylene Ethylene Propylene Diene Monomer
1J	11698	Safety overflow ball	Polypropylene
1K	11697	Safety overflow cage	Polypropylene
<b>JARS ONLY</b>			
2	17630	100 ml jar only	Polycarbonate
	11731	150 ml jar only	Polycarbonate

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### ANTI-BACTERIAL FILTERS FOR SAFETY JARS

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• **Material:**

- Filter: borosilicate fibreglass + laminated polyester
- Plastic housing: polypropylene

• **Availability:**

Ref.: 11813  
Tube of 10 plastic filters  
for current safety jar for  
RVTM2 vacuum regulator

Ref.: 11818  
(Box of 100 paper filters  
for previous safety jar for  
RVTM vacuum regulator)