SWITCH / FLOW-SWITCH

SWITCH & FLOW-SWITCH

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The Switch and the Flow-Switch can be screwed onto all flow-meters. They enable you to direct the gas flow either towards an outlet tubing nipple or a humidifier. The user can reach the requested position without removing the nipple or the humidifier.

Manufacturer:

TECHNOLOGIE MEDICALE
101, rue Vaillant Couturier - BP 46
93136 Noisy-le-Sec Cedex
FRANCE
Ph.: 33 (0)1 48 45 58 95
Fax: 33 (0)1 49 42 90 21

• **Classification rule and EC certificate registration No.:**

*- Active device of class IIa non sterile - according to the classification of the annex IX of the european directive 93/42/EEC and 2007/47/EEC, chapter 3 rule 11.
- HD 60004355 0001
- CE 0197 – TÜV RHEINLAND
- Directive 93/42/EEC and 2007/47/EEC, Annex II, Article 3*

• **UMDNS code and product group:**

11748: "Flowmeter gas"

• **GMDN code and product group:**

35074: "A connecting device, usually a small part used to join two or more pieces of apparatus or device components, or make them compatible so that they mate together."

• **Applied standard to the products:**

EN ISO 15002: 2008 - Flow-metering devices for connection to terminal units of medical gas pipeline systems.

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• **Dimensions :**

• **Versions:**

Switch: Model with knob
Flow-Switch: Model with ring

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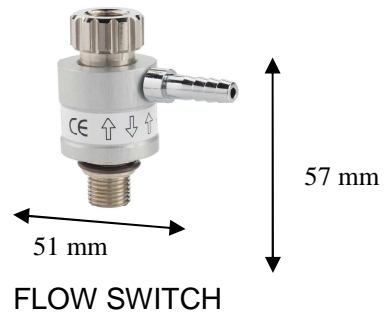
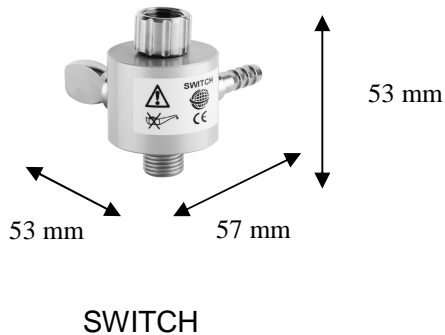
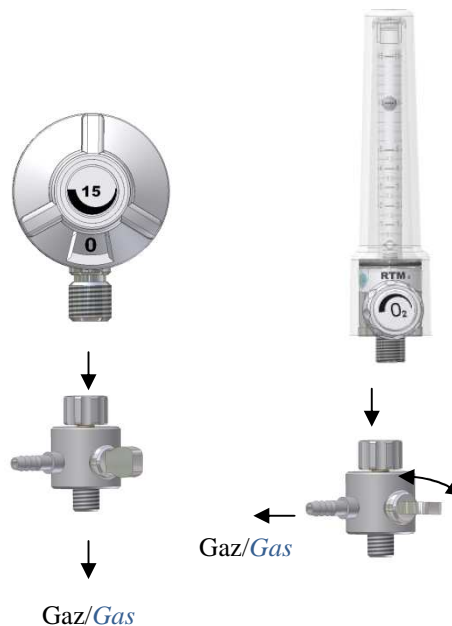
• **Connection:**

Switch:

Connect the Switch to the flowmeter outlet. By giving the central knob a quarter turn, you can choose to orientate the flow either to the outlet tubing nipple or to a humidifier.

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
Flow-Switch:

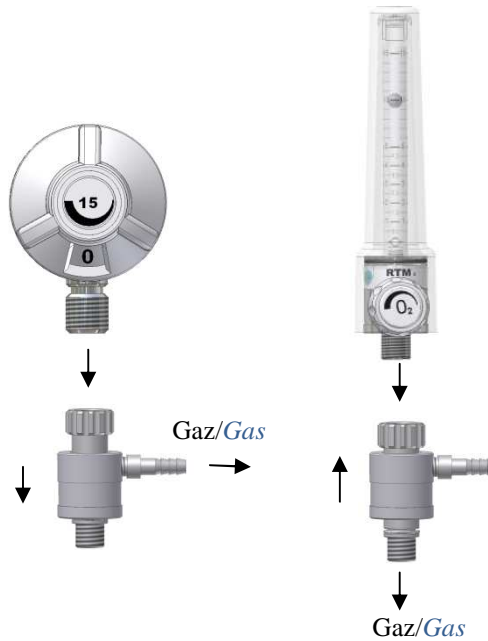Connect the Flow-Switch to the flowmeter outlet.
To direct the gas flow to the outlet tubing nipple just pull the ring down.
To direct the gas flow to a humidifier pull the ring up.

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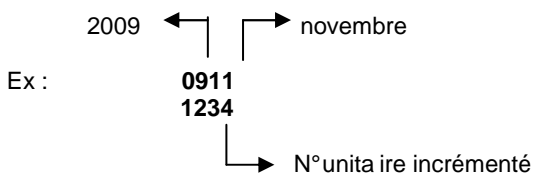
• **Packing / delivery:**

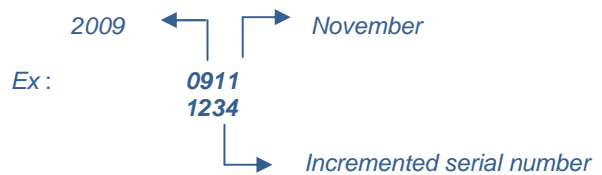
Switch and Flow-Switch are delivered in an individual sealed plastic bag with instructions for use.

• **Traceability:**

A serial number is engraved on each device body for identification and traceability.

8 digits serial number:

1st and 2nd digits = 2 last numbers of manufacturing year
3rd and 4th digits = manufacturing month
3 or 4 last digits = incremented serial number

• **Cleaning:**

Clean the outside of the device with water and soap. Rinse and dry.

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• **Maintenance:**

Once a year, check the seals and change them if necessary. Lubricate the body **exclusively with oxygen compatible grease or oil.**

• **Warranty :**

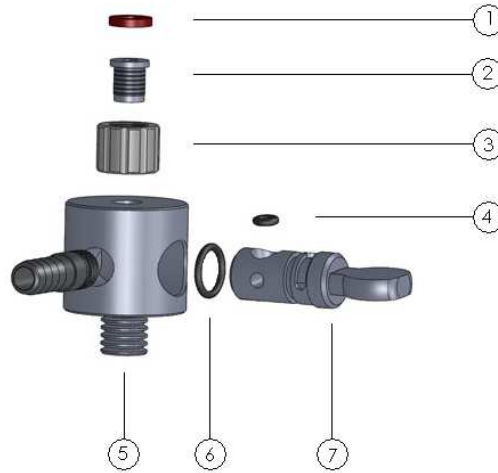
1 year warranty, parts and service, except in case of damages or accidents due to carelessness, misuse and bad supervision or maintenance.

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