CE 0197

CCO TM

HUMIDIFICATEUR
HUMIDIFIER
HUMIDIFICADOR
BEFEUCHTER
UMIDIFICATORE
HUMIDIFICADOR
FUKTARE
BEVOCHTIGER
ОВЛАЖНИТЕЛ
ΥΓΡΑΝΤΗΡΑΣ
PÁRÁSITÓ
Drėkintuvas
UMDATUR
NAWILŻACZ
UMIDIFICATOR
ZVLHČOVAČ
VLAŽILEC
ZVLHCOVAC
مرطب

F	Notice d'instructions.....
GB	Instructions for use.....
ES	Folleto de instrucciones.....
DE	Bedienungsanleitung.....
IT	Avvertenze e istruzioni.....
PT	Manual de Instruções.....
SE	Bruksanvisning.....
NL	Gebruiksaanwijzing.....
BG	Инструкции за експлоатация...
GR	Οδηγίες χρήσης.....
HU	Használati útmutató.....
LT	Naudojimo instrukcija.....
MT	Ara n-nota ta' struzzjonijiet.....
PL	Instrukcja obsługi.....
RO	Manual de Instrucțiuni.....
SK	Návod na použitie.....
SL	Navodila za uporabo.....
CZ	Návod k použití.....
AR	ارشادات الاستخدام.....

HUMIDIFICATEUR CCO TM 250 ml

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Fabricant : TECHNOLOGIE MEDICALE - 101, rue Vaillant Couturier - BP 46
93136 Noisy-le-Sec cedex - France
Tel. : +33 (0)1 48 45 58 95 - Fax : +33 (0)1 49 42 90 21

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Purpose: device meant to humidify a medical gas.

Storage: between -10°C / +40°C in a dry & clean area.

Connection: screw the nut onto the flowmeter outlet thread. Fit hard the tubing sleeve onto the notched nipple.

The table below indicates the thread of the device according to the colour of the instructions for use:

colour	blue	pink	green	salmon-pink	yellow
thread	12x125	1/4 G	1/2 BS	9/16"	3/8 G

Features:

- Consumption: 8 ml/h of water for a 10 l/min flow rate at 23°C.

- Useful capacity of humidification: 160 ml.

Materials:

Bottle: polysulfone

Diffuser: polyethylene - O-ring: nitrile

Tubing and gasket: silicone - Cover: polypropylene

Instructions for use: the bubbling bottle should only contain sterile water. Pay attention to the MIN / MAX marks.

Cleaning / Maintenance:

- the single-use diffuser (part A) must be changed after each patient and after each cleaning of the humidifier.

- the silicone tubing can be re-used after cleaning and disinfection.

Cleaning / Disinfection / Sterilization:

Polysulfone withstands autoclave up to 134°C (273°F).

Always check the compatibility of polysulfone with the disinfecting products.

Disinfecting products containing the following components are not compatible : amphoteric components, glutaraldehyde, formaldehyde, glyoxal, succinic acids .

Cleaning products for floors and surfaces must be proscribed.

Indeed the polysulfone may react with residues of these products.

Recommended protocol: unscrew the whole product, throw away the diffuser, dump into a decontaminating solution, wash, rinse, soak during 1 min. in clear water, dry and disinfect or sterilize, put a new diffuser and re-use.

As per the European Directive 93/42/CEE dated June 14th 1993, TECHNOLOGIE MEDICALE must be immediately notified of any incident or risk of incident.

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