BOCAL DE RECUEIL REUTILISABLE

COLLECTION JAR REUSABLE

F	Notice d'instructions
GB	Instructions for use
D	Bedienungsanleitung
E	Folleto de instrucciones
I	Avvertenze e istruzioni
P	Manual de Instruções
BG	Инструкции за експлоатация
GR	Οδηγίες χρήσης
H	Használati útmutató
LT	Naudojimo instrukcija
MT	Ara n-nota ta' struzzjonijiet
NL	Gebbruksinstructies
PL	Instrukcja obsługi
RO	Manual de Instrucțiuni
SK	Návod na použitie
SL	Navodila za uporabo
SE	Bruksanvisning
CZ	Návod k použití
AR	إرشادات الاستخدام

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1. Presentation
2. Versions
3. Technical data
4. Instructions for use
5. Symbols
6. Safety advice
7. Cleaning and disinfection
8. Storage
9. Maintenance
10. Guarantee

1. Presentation

Medical device meant to collect secretions and other substances during operations of medical suction. Do not use the device alone, but with a complete suction system, including a mechanical or electrical suction device.

2. Versions

Available versions:

- Capacities: 1 L., 2 L. or 4 L.
- Material: polysulfone or polycarbonate
- Cover: screw-fit metal lid, pressure-fit plastic lid, pressure silicone ring lid with metal plate, screw-fit plastic lid

3. Technical data

- In accordance with EN ISO 10079-3: 2009 standard
- Unit: milliliter (ml)
- Anti overflow safety system integrated to covers
- Moulded hanger on 2 L. jars
- Reusable
- Operating temperature: 20°C (= 68°F).

4. Instructions for use

- Make sure the cover is safely set up on the jar
- Make sure the "vacuum" and "patient" nipples are screwed tight enough
- Connect the jar with suction hose (Ø 6 to 8 mm for 1 and 2 L. jars - Ø 8 to 10 mm for 4 L. jars)
 - o "patient" nipple: towards the suction catheter
 - o "vacuum" nipple: towards the suction controller
- Check the tightness of the complete system (for the pressure-fit plastic lid, check that the seal is correctly positioned in the groove, and open the lip seal)

Once the operation is done, make sure the suction is stopped and blow off the system before manipulating jar and/or cover.

5. Symbols

See instructions for use

6. Safety advice

- Always make a working test prior to any use with patients.
- Make sure a filter is included in the suction system (with collection jar or suction controller)
- Do not manipulate a vacuum-connected device.

7. Cleaning and disinfection

GB

Caution, jar stiffness can be altered after repeated cleanings in washing-machine.

1. Disassemble jar and cover parts
2. Immerse components in a decontaminating or pre-disinfecting solution (respecting manufacturers protocols concerning soaking, rinsing and drying times)
3. Clean (do not use abrasive items), rinse thoroughly, and let soak for 1 minute in clean water, then dry
4. Sterilize or disinfect, respecting manufacturers protocols (disassembled parts, jar in vertical position, upside down, do not place anything on top of jars)
5. Put the parts back together

Polysulfone withstands autoclave up to 134°C (273°F).

Polycarbonate withstands autoclave up to 121°C (249°F).

Caution, compatibility of products used with polycarbonate must be checked.

For information, products containing amphoteric components, glutaraldehyde, formaldehyde, glyoxal, succinic acids are not compatible with polycarbonates. Also avoid solutions for floors and surfaces.

8. Storage

- Between -10 and +40°C (betw. 14-104°F) in a dry and clean place.
- Keep the device in its plastic bag as long as possible.

9. Maintenance

Once a year, check the tightness of the jar and the state of the gaskets.

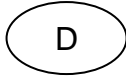
Spare parts		Use exclusively spare parts from
- Gaskets		TECHNOLOGIE MEDICALE

10. Guarantee

1 year warranty (parts and service), except in case of normal wear, damages or accidents due to carelessness, misuse and bad supervision or maintenance.

Contractual lifetime: 10 years

*As per the European Directive 93/42/CEE dated June 14th, 1993,
TECHNOLOGIE MEDICALE must be immediately notified of any incident or risk of incident*

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