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DEBITMETRE PRECALIBRE DEBSON TM2

DEBSON TM2 FLOWMETER WITH PRE-ADJUSTED FLOWRATES

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Medical device meant to adjust the flow of a gas. To be connected on a medical gas distribution network using a direct probe or a rail mounting system.

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• **Classification rule and CE certificate registration No.:**

- Active device of class IIa non sterile - according to the classification of the annex IX of the european directive 93/42/EEC and 2007/47/EEC, chapter 3 rule 11.
- CE 0197 – TÜV RHEINLAND
- HD 60020766 0001
- Directive 93/42/EEC and 2007/47/EEC, Annexe II, Article 3

• **UMDNS code and product group:**

11748 : "Flowmeter gas"

• **GMDN code and product group:**

37132: "A device used for the administration of pure oxygen (O₂). It is used to treat very ill patients, e.g., those suffering cardiac infarction, having insufficient oxygen uptake, or experiencing postoperative relapse. The device can, e.g. consist of a flowmeter with or without humidification, have suction possibilities, and be connected to a central gas supply or have its own tank."

• **Applied standards to the products:**

EN ISO 15002: 2008 – Flowmetering devices for connection to terminal units of medical gas pipeline systems.

EN ISO 7396-1: 2007 – Medical gas pipeline systems - Part 1: Pipelines systems for compressed medical gases and vacuum.

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• **Available gases**: Oxygen (O₂) and Medical Air

• **Working pressure**: 4.5 bar +/-0.5

• **Filter porosity**: 35 microns

• **Unit of measurement**: litre per minute (l/min)

• **Reading accuracy**: ± 10 % of the read value

• **Working temperature**: 23°C

• **Available flowrates**:

1 l/min:	0	0.1	0.2	0.3	0.4	0.5	0.6	0.7	0.8	1
5 l/min:	0	0.5	0.8	1	1.5	2	2.5	3	4	5
15 l/min:	0	1	2	3	4	5	6	9	12	15
25 l/min:	0	0.5	1	2	3	4	6	8	15	25
50 l/min:	0	5	10	15	20	25	30	35	40	50

• **Threaded outlets**:

- 12x125 M
- 9/16" M
- 1/4G M
- 1/2"BS F

• **Connection to standardized wall outlet**:

Either directly with a direct probe,
Or with a complete rail mounting system (rail adaptor, metal sliding piece, 1.5 m hose and crimped remote probe).

• **Weight without probe**: 250 g.

• **Dimensions without probe**: Diameter 40 mm

• **Principe**:

The principle of flowmeters with pre-adjusted flowrates lies in the variation of the flow-holes diameter.

A circular disc has several holes whose diameters vary increasingly. The rotation of the disc changes the hole enabling the flow of gas and changes the flowrate. The larger the hole diameter is, the higher is the flowrate.

Thanks to this design, flowrates remain accurate, stable and time-resistant.

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Ne pas immerger.

• **Technical data:**

Flowmeter with pre-adjusted flowrates: stable and accurate flowrates.

Can be used in any orientation for a perfect adaptation to the medical needs, especially for emergency care.

Flow control knob with a ratchet mechanism avoiding accidental flowrate change and preventing from standing between 2 flowrates.

Control knob with large wings facilitating the grip and including a twin display of the selected flowrate (front & top).

Control knob with a wide transparent window, for a good reading of the selected flowrate as well as the previous and the next ones.

Direction of rotation: counterclockwise.

Standard model delivered with a stop system to avoid switching directly from 0 to the maximum flow and conversely. Model without Stop system available for emergency care.

Double filtration: filters at the inlet and at the outlet for optimal safety of both the patient and the device.

Metal parts made of nickel-plated brass.

• **Use:**

Connect the device into the appropriate wall outlet.

Turn the flow control knob to the left until you reach the requested flowrate.

• **Traceability:**

A unit serial number is engraved on each flowmeter body for identification and traceability.

8 digits serial number

1st and 2nd digits = 2 last numbers of manufacturing year

3rd and 4th digits = manufacturing month

4 last digits = incremented serial number

• **Cleaning:**

Clean the outside of the device with water and soap. Rinse and dry. If using disinfecting products please check their compatibility with plastics.

Do not lay under water.

DEBITMETRE PRECALIBRE / FLOWMETER WITH PRE-ADJUSTED FLOWRATES

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• **Maintenance:**

The maintenance depends on the use of the device but a revision must be carried out every 5 years maximum.

Only the manufacturer, or an authorized person can operate on inner spare parts.

• **Safety:**

Do not use close to any ignition point.

Do not bring the device into contact with greasy products.

• **Packing / delivery:**

Delivered in an individual sealed plastic bag with instructions for use.

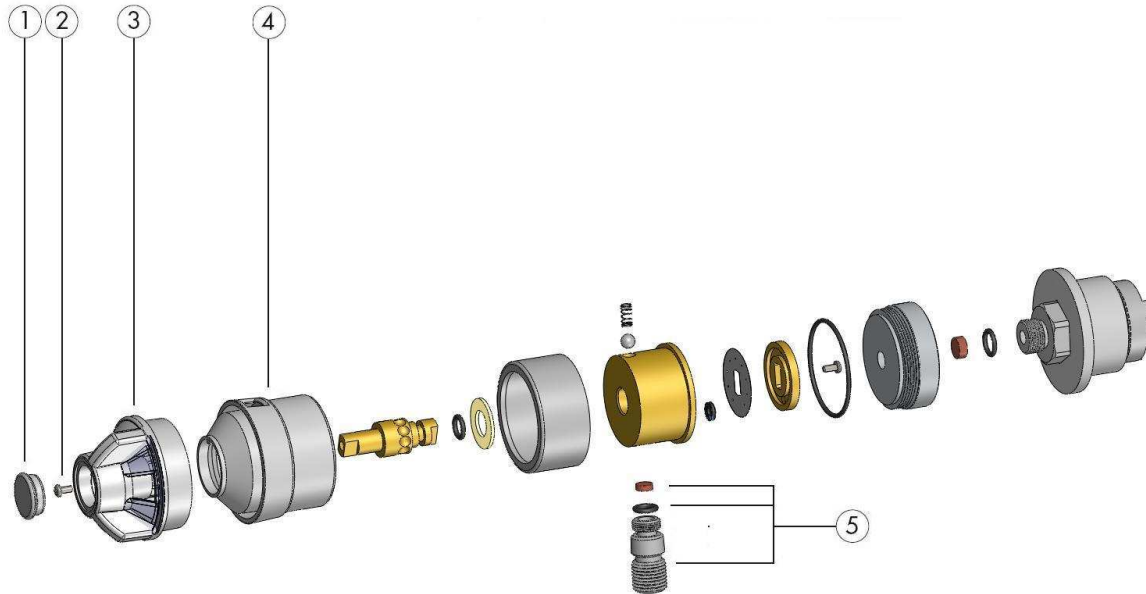
• **Warranty:**

1 year warranty (parts and service), except in case of damage or accidents due to carelessness, misuse and bad supervision or maintenance.

• **Contractual life time:** 10 years

DEBITMETRE PRECALIBRE / FLOWMETER WITH PRE-ADJUSTED FLOWRATES

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N°	Ref.	Description	Material
FRONT DISC			
1	18503	White disc O ₂ 01 l/min	Polypropylene
	18505	White disc O ₂ 05 l/min	Polypropylene
	18504	White disc O ₂ 15 l/min	Polypropylene
	18514	White disc O ₂ 25 l/min	Polypropylene
	18506	White disc O ₂ 50 l/min	Polypropylene
	18508	Black disc AIR 01 l/min	Polypropylene
	18507	Black disc AIR 05 l/min	Polypropylene
	18509	Black disc AIR 15 l/min	Polypropylene
	18513	Black disc AIR 25 l/min	Polypropylene
	18510	Black disc AIR 50 l/min	Polypropylene
2	11030	Tightening screw	Steel
WHEEL			
3	17723	Wheel 01 l/min	Xenoy
	17729	Wheel 05 l/min	Xenoy
	17722	Wheel 15 l/min	Xenoy
	18527	Wheel 25 l/min	Xenoy
	17724	Wheel 50 l/min	Xenoy
COMPLETE TUBE WITH STOP SYSTEM			
4	17715	01 l/min tube with stop system	Xenoy
	17717	05 l/min tube with stop system	Xenoy
	17714	15 l/min tube with stop system	Xenoy
	18525	25 l/min tube with stop system	Xenoy
	17716	50 l/min tube with stop system	Xenoy
COMPLETE OUTLET			
5	11052	Complete outlet 12x125M	Nitrile + bronze + brass
	11054	Complete outlet 9/16" M	Nitrile + bronze + brass
	11056	Complete outlet 1/4G M	Nitrile + bronze + brass
	16921	Complete outlet 1/2"BS F	Nitrile + bronze + brass