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DEBITMETRE A BILLE RTM3 RTM3 FLOWMETER WITH FLOATING BALL

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Medical device meant to adjust and measure the flow of a gas. To be connected on a medical gas distribution network.

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• **Classification rule and CE certificate registration No.:**

- Active device of class IIa non sterile - according to the classification of the annex IX of the european directive 93/42/EEC and 2007/47/EEC, chapter 3 rule 11.
- CE 0197 – TÜV RHEINLAND
- HD 60020766 0001
- Directive 93/42/EEC and 2007/47/EEC, Annexe II, Article 3

• **UMDNS code and product group:**

11748: "Flowmeter gas"

• **GMDN code and product group:**

37132: "A device used for the administration of pure oxygen (O₂). It is used to treat very ill patients, e.g., those suffering cardiac infarction, having insufficient oxygen uptake, or experiencing postoperative relapse. The device can, e.g. consist of a flowmeter with or without humidification, have suction possibilities, and be connected to a central gas supply or have its own tank."

• **Applied standards to the products:**

EN ISO 15002: 2008 - Flow-metering devices for connection to terminal units of medical gas pipeline systems.

EN ISO 7396-1: 2007 - Medical gas pipeline systems - Part 1: Pipelines systems for compressed medical gases and vacuum.

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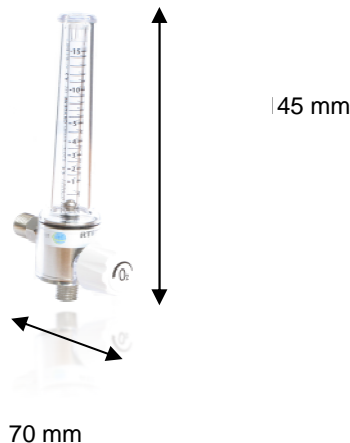
• **Filter porosity**: 35 microns

• **Unit of measurement**: litre per minute (l/min)

• **Reading accuracy**: ± 10 % of the read value
Value to be read in the middle of the ball

• **Maximun flowrate (out of scale)**: 40 l/min

• **Working temperature**: 23°C

• **Different flowrates available**:

- **1.5 l/min**

Scale graduated every 0.1 l/min

- **5 l/min (expanded to 1.5 l/min)**

Scale graduated every 0.25 l/min from 0.5 l/min up to 2l/min,
then graduated every 0.5 l/min

- **15 l/min (expanded to 5 l/min)**

Scale graduated every 0.5 l/min from 0.5 l/min up to 5 l/min,
then graduated every 1 l/min

- **30 l/min**

Scale graduated every 1 l/min from 5 l/min up to 30 l/min.

• **Several threaded outlets available**:

- 12x125 M

- 9/16" M

- 1/4 G M

- 1/2" BS F

• **Connection to the standardized wall outlet**:

Either directly with a direct probe,

Or with a complete rail mounting system (rail adaptor, metal
 sliding piece, 1.5 m hose and crimped remote probe).

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• **Features:**

Flowmeter with compensated pressure ensuring a perfect flowrate stability and accuracy. The scale works with the same pressure than the one of the network and is not sensitive to back-pressure.

Expanded scale providing higher reading accuracy for lower flowrates.

Type of "cartridge" knob, extremely reliable, easy-to-clean and ensuring an optimized adjustment. Perfect tightness of the knob thanks to its polyamide seat. The knob was tested under pressurized working conditions during more than 10 000 cycles (which correspond to 5 openings / closings every day during more than 5 years) without showing any leakage after closing. This allows us to guarantee the perfect tightness and the closing of the knob during 5 years without any leakage.

Monoblock scale cover made of polycarbonate for a better sturdiness and a higher shock-resistance. Patients have no access to the scale. A tool (key no.14) is necessary to reach the measuring scale. The inopportune unscrewing of the monoblock scale cover which completely covers the body of the flowmeter is not possible. The self-adhesive identification ring is protected against tarnishing and wear & tear.

A metallic sieve located at the inlet of the flowmeter protects the device against impurities of the gas network and thus ensures the protection of the patient. Filter porosity : 35 microns.

The devices are individually and permanently stamped to ensure their identification and traceability.

Body made of nickel-plated brass.

Easy cleaning thanks to its monoblock scale cover without any sharp edge.

Top stopper of the scale is fixed on the tube to facilitate the maintenance operations while trapping in the floating ball inside the tube.

• **Use:**

For a correct use, the flowmeter with floating ball must be fixed in a vertical position.

• **Adjustment:**

Turn the knob anti-clockwise to increase the flow and adjust the requested flowrate. Reading in the middle of the ball.

• **Closing:**

Turn the knob clockwise until tightness. Do not close hard to avoid wear.

Caution! Make sure no water enters the flowmeter during transportation with a humidifier. Always remove the humidifier before any operation.

DEBITMETRE A BILLE RTM3 / RTM3 FLOWMETER WITH FLOATING BALL

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• **Traceability:**

A unit serial number is engraved on each flowmeter body for identification and traceability.

RTM3

8 digits serial number + one letter

1st and 2nd digits = 2 last numbers of manufacturing year
3rd and 4th digits = manufacturing month
last digits = incremented serial number

Ex : 01 11 1234 A

2001 ← 11 → November
← 1234 → Incremented serial number
← A → one letter at the end

• **Cleaning:**

Clean the outside of the device with water and soap. Rinse and dry. Make sure the products used are compatible with plastics.

• **Maintenance:**

Check every 1 to 3 years depending on the use:

- Inspection of flow accuracy
- Inspection of leakage

In case of damage or malfunction, send the device back to the manufacturer or to a registered provider.

• **Packing / delivery:**

Delivered in an individual sealed plastic bag with instructions for use.

• **Warranty:**

Warranty of the closing of the knob during 5 years without any leakage.

1 year warranty, parts and service, except in case of damages or accidents due to carelessness, misuse and bad supervision or maintenance.

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