## DEBSON TM2

F	Notice d'instructions
GB	Instructions for use
D	Bedienungsanleitung
E	Folleto de instrucciones
I	Avvertenze e istruzioni
P	Manual de Instruções
BG	Инструкции за експлоатация
GR	Οδηγίες χρήσης
H	Használati útmutató
LT	Naudojimo instrukcija
MT	Ara n-nota ta' struzzjonijiet
NL	Gebruiksaanwijzing
PL	Instrukcja obsługi
RO	Manual de Instrucțiuni
SK	Návod na použitie
SL	Navodila za uporabo
SE	Bruksanvisning
CZ	Návod k použití
AR	إرشادات الاستخدام

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1. Presentation
2. Versions
3. Technical data
4. Instructions for use
5. Symbols
6. Safety advice
7. Cleaning and disinfection
8. Storage
9. Maintenance
10. Guarantee

## 1. Presentation

Medical device meant to be connected on a pressure regulator or on a medical gas distribution system, in order to adjust the gas flow.

## 2. Versions

Available versions:

- Gas: oxygen (O<sub>2</sub>) or medical air
- Inlet pressure: 4,5 bar (+/- 0,5 bar).
- Connecting probe: according to standards NF S 90-116 - BS 5682 - DIN 13260  
US Ohmeda Diamond - NORDIC - UNI.
- Direct connection or mounting on a rail (Hose + crimped remote probe + rail clamp).
- Threaded outlet: 12x125 M - 9/16" M - 1/4G M - 1/2"BS F.
- Delivered in standard with stopped system.
- Flow rates: 1 - 5 - 15 - 25 - 50 l/min (each version supplies 9 flow rates):

<b>1 l/min</b>	:	0	0.1	0.2	0.3	0.4	0.5	0.6	0.7	0.8	1
<b>5 l/min</b>	:	0	0.5	0.8	1	1.5	2	2.5	3	4	5
<b>15 l/min</b>	:	0	1	2	3	4	5	6	9	12	15
<b>25 l/min</b>	:	0	0,5	1	2	3	4	6	8	15	25
<b>50 l/min</b>	:	0	5	10	15	20	25	30	35	40	50

## 3. Technical data

- In accordance with EN ISO 15002: 2008 standard.
- In accordance with EN ISO 7396-1: 2007 standard.
- Unit: litre per minute (l/min).
- Accuracy of measures: 10 % of the value.
- Filter: 35 µm porosity.
- Unit serial number: engraved on the device body.
- Influence of inlet pressure: check the inlet pressure. If the inlet pressure is not within the margin displayed on the device, flow may be out of authorized standard range.
- Working temperature: 23°C (73,4°F).

## 4. Instructions for use

- Make sure the device is undamaged.
- Make sure the flow rate is on 0.
- Connect the device on the wall outlet or on the pressure regulator.
- Connect patient tubing to the flowmeter or to the humidifier.
- Turn the front knob to the left and select the flow rate to be administered.
- Turn the adjustment the wheel on the front to the left until the desired pressure is displayed.

It is recommended to remove any device from the gas supply when not used.

## 5. Symbols

Do not use grease or oil

See instructions for use

GB

## 6. Safety advice

- Do not smoke near the device.
- Do not use the device near ignition sources.
- Do not use oil or grease with the device.
- Do not handle the device with greasy hands.
- Do not disassemble any device when under pressure.

## 7. Cleaning and disinfection

Clean the exterior of the device with water and soap. Rinse and dry. If using disinfecting products please check their compatibility with plastics (see specification).

Do not use floor cleaning products.

## 8. Storage

Between -20 and +60°C (-4 and 140°F) in a dry and clean place.  
Keep the device in its plastic bag as long as possible.

## 9. Maintenance

Check-up of the wear of gaskets and lubrication with an oxygen-compatible product.  
In case of damage or malfunction, return the device to the manufacturer or to a registered provider.

A complete maintenance including control of flow rates should be carried out according to device use, but at least once **every 5 years**.

<b>Spare parts</b>	<u>Use exclusively spare parts from TECHNOLOGIE MEDICALE</u>
- Gaskets	
- Probe	

## 10. Guarantee

1 year warranty (parts and service), except in case of damage or accidents due to carelessness, misuse, bad supervision or maintenance.

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