CE 0197

RTM3

F	Notice d'instructions
GB	Instructions for use
D	Bedienungsanleitung
E	Folleto de instrucciones
I	Avvertenze e istruzioni
P	Manual de Instruções
BG	Инструкции за експлоатация
GR	Οδηγίες χρήσης
H	Használati útmutató
LT	Naudojimo instrukcija
MT	Ara n-nota ta' struzzjonijiet
NL	Gebbruksinstructies
PL	Instrukcja obsługi
RO	Manual de Instrucțiuni
SK	Návod na použitie
SL	Navodila za uporabo
SE	Bruksanvisning
CZ	Návod k použití
AR	إرشادات الاستخدام
CN	流量表

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1. Presentation
2. Versions
3. Technical data
4. Instructions for use
5. Symbols
6. Safety advice
7. Cleaning and disinfection
8. Storage
9. Maintenance
10. Guarantee

1. Presentation

Medical device used to **adjust and measure the flow of a gas**. It should be connected to a source of pressurized gas on the wall

2. Versions

Available versions:

- Gas: oxygen (O₂) or medical air
- Inlet pressure: 4 bar (+1 / -0)
- Connecting probe: according to standards NF S 90-116 - BS 5682 - DIN 13260 - US Ohmeda Diamond - NORDIC - UNI
- Direct connection or mounting on a rail (Hose + crimped remote probe + rail clamp)
- Threaded outlet: 12x125 M - 9/16" M - 1/4G M - 1/2"BS F
- Flow rates available: 1,5 - 5 - 15 - 30 l/min

3. Technical data

- In accordance with EN ISO 15002: 2008 standard
- In accordance with EN ISO 7396-1: 2007 standard
- Unit: litre per minute (l/min)
- Accuracy of measures: 10 % of full scale
- Filter: 35 µm porosity
- Unit serial number: engraved on the device body
- Influence of inlet pressure: check the inlet pressure. If the inlet pressure is not within the margin displayed on the device, flow may be out of authorized standard range
- Maximum flow rate out of scale: 40 l/min
- Flowmeter with compensated pressure (except for paediatric flowmeter: atmospheric pressure)
- Working temperature: 23°C (73,4°F)

4. Instructions for use

- Make sure the device is undamaged
- Make sure the knob is shut
- Connect the device on the wall outlet or on the pressure regulator, in vertical position
- Connect patient tubing to the flowmeter or to the humidifier
- Turn the front regulation knob to the left and thanks to the graduated scale select the gas flow to be delivered (reading in the middle of the ball)

It is recommended to remove any device from the gas supply when not used

5. Symbols

Do not use grease or oil

See instructions for use

GB

6. Safety advice

- Do not smoke near the device
- Do not use the device near ignition sources
- Do not use oil or grease with the device
- Do not handle the device with greasy hands
- Do not disassemble any device when under pressure
- Make sure the device remains in vertical position
- Caution! Do not let water penetrate into the flowmeter during the transportation with a humidifier; always unscrew the humidifier before handling

7. Cleaning and disinfection

- Clean the exterior of the device with water and soap. Rinse and dry. If using disinfecting products please check their compatibility with plastics (see specification)
- Do not use floor cleaning products

8. Storage

Between -20 and +60°C (betw. -4 and 140°F) in a dry and clean place
Keep the device's packaging

9. Maintenance

Check every 1 to 3 years depending on the device use:

- Inspection of flow accuracy
- Inspection of tightness

In case of damage or malfunction, ship the device back to the manufacturer or to a registered provider

Spare parts	<u>Use exclusively</u> <u>TECHNOLOGIE MEDICALE</u> <u>spare parts</u>
- Knob	
- Gaskets	
- Probe	

10. Guarantee

- 1 year warranty (parts and service), except in case of damage or accidents due to carelessness, misuse, bad supervision or maintenance
- The closing of the regulation knob is guaranteed during 5 years without any leakage
- Contractual lifetime: 10 years

*As per the European Directive 93/42/CEE dated June 14th, 1993,
TECHNOLOGIE MEDICALE must be immediately notified of any incident or risk of incident*

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