CE 0459

# DEBSON TM2

FR	Notice d'instructions
EN	Instructions for use
DE	Bedienungsanleitung
ES	Folleto de instrucciones
PT	Manual de Instruções
IT	Avvertenze e istruzioni
SV	Bruksanvisning
NL	Gebruiksaanstructies
AR	إرشادات الاستخدام

BG	Инструкции за експлоатация
CS	Návod k použití
DA	Bruksanvisning
EL	Οδηγίες χρήσης
FI	Käyttöohjeet
HU	Használati útmutató
LT	Naudojimo instrukcija
MT	Struzzjonijiet għall-użu
PL	Instrukcja obsługi
RO	Manual de Instrucțiuni
SK	Návod na použitie
SL	Navodila za uporabo

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## 1. Presentation

The DEBSON TM2 is a medical device designed to control gas flow. It connects to a pressurised-gas wall fitting or to a pressure regulator output.

User profile:

Users are medical staff: doctors, nurses.

## 2. Versions

DEBSON TM2 is available in the following versions:

- Gas: oxygen (O<sub>2</sub>) or medical air.
- Supply pressure: 4.5 bar (+/- 0.5 bar).
- Connector nozzle: compliant with NF S 90-116 - BS 5682 - DIN 13260  
US Ohmeda Diamond - NORDIC - UNI...
- Direct connection or via a rail-mounting system (hose + crimped remote probe + rail clamp).
- Threaded outlet: 12x125 M - 9/16" M-1/4G M-1/2" BS F.
- Single or double version.
- Delivered as standard with a stop system.
- Flow ranges (9 flow positions):

	(Flows expressed in l/min)										
<b>0 - 1 l/min</b>	:	0	0.1	0.2	0.3	0.4	0.5	0.6	0.7	0.8	1
<b>0 - 5 l/min</b>	:	0	0.5	0.8	1	1.5	2	2.5	3	4	5
<b>0 - 15 l/min</b>	:	0	1	2	3	4	5	6	9	12	15
<b>0 - 25 l/min</b>	:	0	0.5	1	2	3	4	6	8	15	25
<b>0 - 50 l/min</b>	:	0	5	10	15	20	25	30	35	40	50

## 3. Technical data

- Compliant with NF EN ISO 15002.
- Class IIa medical device.
- Measurement unit: litre per minute (l/min).
- Measurement accuracy:  $\pm 30\% \leq 1,5$  l/min ;  $\pm 20\% > 1,5$  l/min.
- Continuous flow between two positions.
- Inlet and outlet filters: porosity 35  $\mu$ m.
- Individual serial number with month and year of manufacture
- Operating temperature: 23°C.

## 4. Instructions for use

- Check that the device is in good working order.
- Check that the flow is on 0.
- Connect the device to the wall fitting or the pressure regulator outlet.
- Screw on a humidifier if necessary, or a nipple to link it to the tubing of the therapy equipment.
- Turn the adjustment button or wheel on the front to the left until desired flow is displayed.
- Attach the tubing to the patient at the last stage.
- To disconnect: close the flow meter (position 0 l/min), remove the tubing from the patient, disconnect the humidifier if used, then disconnect the flow meter from the wall fitting or pressure regulator outlet.

## 5. Symbols

	Do not use grease, lubricant or oil
	See instructions for use
YYMMxxxx	Serial number engraved on the device: YY: Year of manufacture, MM: Month of manufacture, xxxx: Unit number
	Manufacturer

## **6. Safety advice**

- To avoid any risk of fire or explosion:
  - ✓ Do not smoke in the vicinity.
  - ✓ Do not use near a source of ignition.
  - ✓ Do not allow the device to come into contact with greasy substances.
  - ✓ Do not handle the device with greasy hands.
  - ✓ Do not disassemble a pressurised device.
- Do not constrict the outlet of the medical device.
- Influence of pressure: Caution, check the supply pressure to the device. If the inlet pressure is not within the range shown on the device, there is a risk that the flow may fall outside of the authorised standard range.
- Influence of temperature: Caution, variations in room temperature (between 0°C and 40°C) can alter the accuracy of the flows delivered.
- Do not disassemble the device's inlet or output connectors.
- On this type of device, there is no indicator showing gas flow.
- The flow should be adjusted to one of the click-set positions. Never position the click-set button between two flows.

## **7. Cleaning and disinfection**

- Disconnect the device from the gas supply. Clean the outside of the device with a clean, slightly damp cloth. Leave the device to dry. If using disinfectants, check their compatibility with the plastic (see technical sheet).
- Do not immerse the device.
- Do not use surface disinfectants.

## **8. Storage**

Between -20 and +60°C in a clean and dry place.  
If possible, store in original packaging.

## **9. Maintenance**

No preventive maintenance is required.

Only Technologie Médicale authorised personnel can disassemble the device.

Technologie Médicale advises you to carry out a yearly service of the device:

- ✓ Check the connection with the wall fitting or pressure regulator outlet.
- ✓ Check that gas is correctly released at every flow adjustment click setting.
- ✓ Check that there are no leaks when the button is at position 0.

## **10. Guarantee**

Device guaranteed for one year (parts and labour), excluding deterioration or accidents due to negligence, improper use or failure to monitor or maintain the device.

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