CE0459

MINIREG

FR	Notice d'instruction
EN	Instructions for use
DE	Gebrauchsweisung
FI	Käyttöohjeet

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1. Presentation

This medical device, when connected to a cylinder of high pressure (HP) medical gas, reduces the pressure so as to deliver a single, pre-set pressure at its outlet. The device's pressure gauge displays the pressure in the cylinder of medical gas.

User profile:

Users are medical staff: doctors, nurses.

2. Versions

- Gas: Oxygen (O₂).
- Inlet connection: AFNOR or DIN.
- Flow rates available for a delivery pressure of 1.6 bar: 2, 4, 5, 6, 8 and 10 l/min.
- Flow rate available for a delivery pressure of 2.75 bar: 10 l/min.
- Threaded outlet: 9/16".

3. Technical data

- | | |
|--|--|
| <ul style="list-style-type: none"> - Complies with standard EN ISO 10524-1: 2006. - Units: bar (pressure) and litres per minute (l/min for flow rate). - Piston-type, single-stage pressure regulator. - Integrated safety valve (pressure relief). - Pressure at inlet: 200 bar max. | <ul style="list-style-type: none"> - Outlet pressure: 1.6 or 2.75 bar (± 0.5 bar). - Ambient temperatures for use: 23°C. - High pressure (315 bar) pressure gauge, class 1.6. - Flow rate accuracy: $\pm 20\%$ - Serial No. engraved on the body (year/month/No.) |
|--|--|

4. Instructions for use

Fitting the pressure regulator to the cylinder

- Check that the device is intact, particularly its inlet fitting.
- Before connecting the pressure regulator to the cylinder, open the gas cylinder's valve slightly, and for a very brief period of time, to blow out any contamination present in the outlet fitting.
- Check that the seal fitted to the pressure regulator's inlet is in place and in good condition.
- Fit the regulator to the cylinder by turning the connector on the regulator until it is hand tight.
- As required, connect a humidifier to the outlet, or fit a tubing nipple to connect it to therapy equipment.
- Open the gas cylinder's outlet valve slowly and gently.
- Run the tubing up to the patient as the final step.

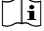
DO NOT USE A SPANNER OR PLIERS

Removing the pressure regulator from the cylinder

- Remove the tubing from the patient.
- Close the valve on the cylinder
- Let the pressurised gas escape from the regulator. The needle on the pressure gauge must indicate 0.
- Disconnect the humidifier if necessary.
- Unscrew (by hand) to remove the device from the cylinder.

WHEN NOT IN USE, THE PRESSURE REGULATOR SHOULD NOT BE LEFT PRESSURISED

5. Symbols

	Do not use grease, lubricant or oil
	Refer to the instructions
YYMMxxxx	Serial No. engraved on the device: YY: Year of manufacture, MM: Month of manufacture, xxxx: Individual number

6. Safety advice

IMPORTANT SAFETY ADVICE REGARDING THE USE OF PRESSURE REGULATORS

All users must be trained in how to handle and use gases:

- Never pressurise/depressurise the device repeatedly over a short period of time.
- Do not use excessive force to close the cylinder's HP valve.
- Do not use the flow of gas from the regulator to power medical equipment.
- Do not create an obstruction to restrict the rate of flow leaving the medical device.
- Effect of temperature: Caution, the flow rate of gas delivered by the device varies as a function of the ambient temperature (between 0°C and 40°C).
- Do not disassemble the device's inlet or outlet connectors.

Standard precautions for oxygen (O₂):

- **Do not smoke.**
- **Do not use near a naked flame.**
- **Do not lubricate.**
Do not apply oily substances (Vaseline, creams) to the patient's face. Handle the equipment with clean, non-greasy hands, and ideally wear clean medical gloves (latex, nitrile).
- Never stand facing the valve outlet when opening; instead ensure that the cylinder is between you and the pressure regulator. Operate the valve at arm's length.
- Ensure that the patient cannot be exposed to a sudden release of gas.
- Do not use aerosol sprays (hairspray, deodorant, etc.), solvents (alcohol, petroleum products, etc.) on the equipment or nearby.
- Check that there are no leaks; if a leak is detected, close the cylinder's valve. Never use a cylinder whose seal is defective.
- Open the cylinder's valve slowly.
- Never use excessive force to open the valve.

Remember that the safe use of pressurised oxygen depends on all users carefully reading all the information provided on the label and in the instructions, and on every incident being reported to the competent authorities.

7. Cleaning and disinfection

- Remove the pressure regulator from the cylinder (refer to the removal procedure).
- Use a disinfectant cleaner for medical devices. Leave to dry before using the device.
- Do not use surface cleaners.
- Do not immerse under water.

8. Storage

- Store at between -20 and +60°C in a clean and dry place.
- Store in its original packaging, if possible.

9. Maintenance

ONLY THE MANUFACTURER, TECHNOLOGIE MEDICALE, CAN WORK ON THE DEVICE.
The manufacturer disclaims all liability if spare parts are used which do not conform with its specification.

Every time before use:

- Check that the device has not been deformed by being knocked or dropped, paying particular attention to the pressure gauge and inlet connector.
- Check that the seal on the connector (for a bullnose valve) or in the yoke (for a pin and index valve) is in place and in good condition.

At least once every 5 years:

- The pressure regulator should be serviced by the manufacturer.
- The due date for the next service is indicated on the device.

Technologie Médicale recommends that the device should be inspected by the user on a yearly basis:

- ✓ Check that the connector (for a bullnose valve) and nut connects correctly to the cylinder
- ✓ Check that the gas flows correctly
- ✓ Check that there are no leaks.

If the equipment is dropped or badly knocked, or if it leaks, arrange for it to be serviced.

Spare parts	Use exclusively TECHNOLOGIE MEDICALE spare parts
Connector seal	

10. Guarantee

- The device is covered by a 1-year warranty (parts and labour), which does not cover damage or incidents caused by negligence, misuse, incorrect supervision or maintenance.
- Contractual service life: 10 years.
- Date when **CE** marking first affixed: 2004

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